

# Easybeat Startup Buttons

Pressure Low +	Demo Mode for sales, training and exhibitions
Pressure Low -	EE Memory Settings – see <a href="http://www.vetsens.com/easybeat/EEmemorySettings.pdf">www.vetsens.com/easybeat/EEmemorySettings.pdf</a>
Pressure High +	Beep off – on respiration pressure line
Pressure High -	Set clock – starting with hh.mm dd.M then yy on left display
SET	Temperature cal – set to 32.0°C then 42.0°C
Alarm button	Temp Typ(e) set to Celsius or Fahrenheit °C or °F
Print button	Respiration rate displayed instead of heart rate